



**VODACLEAR™ Monthly Disposable Contact Lens
(Hioxifilcon A)**

CAUTION: Certain country law restricts this device to sale by or on the order of a licensed practitioner.

1. GETTING STARTED

Your contact lenses should be prescribed by your Eye Care Professional for **daily wear, frequent replacement basis. Cleaning, rinsing and disinfecting is required each time the lens is removed and discarded after the recommended wearing period prescribed by the Eye Care Professional.**

Therefore it is important that you read and understand the information contained in this leaflet and product packaging, along with the VODACLEAR™ Monthly Disposable Contact Lens Patient Instruction Guide & Fitting Guide/Package Insert, as well as any additional restrictions which may be advised to you by your Eye Care Professional.

Please visit www.sglens.com for a copy of the Patient Instruction Guide prior to use.

Strict adherence is necessary for the proper and safe use of contact lenses which in turn allows you to feel and look good with crisp and clear vision.

2. INDICATIONS

The device is designed for daily wear for myopic and hypermetropic visual correction in non-aphakic person with non-diseased eyes with corrective power within the limits of the device power specification.

The product is for single patient use. The monthly frequent replacement lens is intended for daily wear and the lens is to be cleaned, rinsed and disinfected each time it is removed from the eye and replaced every month. However, the qualified Eye Care Professional is strongly encouraged to prescribe the appropriate device replacement schedule based on the ocular response of the patient.

3. DIRECTIONS

Rule of Thumb: Always ensure personal cleanliness and adopt good hygiene methods to ensure safe, comfortable contact lens wear.

a) Preparing the Lens for Wearing

i) Always wash and rinse your hands before handling your lenses. Handle the lenses with your fingertips and avoid contact with fingernails. It is beneficial to keep your fingernails short and smooth.

b) Lens Insertion

i) Develop the habit of working with the same lens first to avoid confusion.

ii) Remove lens from its foil-sealed blister, position it on your index finger and examine it to ensure that it is moist, clean and free from nicks or tears. Do not use if seal is broken, damaged or tampered with.

iii) With the lens on your index finger, use the middle finger of the placement hand to pull down the lower lid. Use the index or middle finger of the other hand to lift the upper lid against the brow. Place the lens on your cornea. While holding this position, look upwards to position the lens properly. Gently release the lids and blink. The lens will center automatically

iv) Repeat the same technique for the other eye.

Once your lenses are properly fitted, you should see clearly and feel comfortable. If you experience poor visual acuity and discomfort, **remove your lenses immediately.**

c) Lens Removal

Look up and slowly pull down your lower lid with the middle of your removal hand and place your index finger on the lower edge of the lens. Gently pinch the lens between the thumb and index finger and remove it.

4. IMPORTANT

a) It is essential that you familiarize and understand the *contraindications, warnings, precautions* and *potential adverse reactions/effects* outlined in the Patient Instruction Guide.

b) Never wear lenses beyond the period recommended by your Eye Care Professional.

c) Always consult your Eye Care Professional when in doubt.

d) If any chemicals of any kind are splashed into the eyes, you should flush eyes immediately with tap water and immediately contact your Eye Care Professional or visit a hospital emergency room without delay.

5. REPORTING OF ADVERSE REACTIONS

All serious adverse experiences and reactions observed in patients wearing or experienced with the lens should be reported to:



SG LENS Pte Ltd
132 Joo Seng Road, #05-01
Singapore 368358
Tel: + 65 6383 2300
Fax: + 65 6339 6609



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

